


Only use HYLO COMOD[®] if the sealing of the folding carton was intact before the first use.

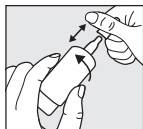

Step 1:


Remove cap before use.

For this purpose, hold the cap just over the bottle. With your other hand, pull away the bottle from the cap with a rotary movement. Before the first application, please turn the HYLO COMOD[®] bottle with its tip down and press onto the bottom of the bottle until the first drop appears at the nozzle. Now the bottle is ready for further use.



Step 2:

Hold the bottle upside down with the thumb on the shoulder of the bottle and the other fingers on the bottom of the bottle.

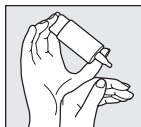


Step 3:

Support the hand holding the COMOD[®]-bottle as shown with the other hand.



Step 4:

Lean your head back slightly, gently pull down your lower eyelid and apply a strong and quick pressure on the middle bottom of the bottle. This activates the mechanism for the release of one drop. Due to the special COMOD[®] valve system, size and speed of each drop are the same no matter how much pressure is applied. Slowly close your eyes to allow the fluid to spread evenly on the eye surface.

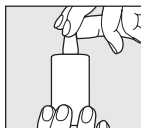

Step 5:

Replace cap immediately after use. Take care that the dropper tip is dry. When applying the fluid, please avoid any contact between the tip of the bottle and your eye or your skin.

The COMOD[®]-system releases 10 ml of solution, which is equivalent to approximately 300 drops. For technical reasons a small amount remains in the bottle at the end of exhaustion.

HYLO COMOD[®] is suitable for long term treatment.

One bottle HYLO COMOD[®] should be used only by the same person.