

## Method of use :

### Hot use :

#### **Recommended Indications for hot use :**

Persistent secretions, inflammation of the palpebral margin and occlusion of the lacrimal duct.

Heat the sealed envelop by putting it into the warm water (aprox 40°C) for 3-5 minutes, after heating, place the wipe on the eyelids for 1-2 minutes with soft massage.



### Cold use :

#### **Recommended Indications for cold use :**

Itching and swelling of the eyelids.

Cool the sealed envelope by putting it into the refrigerator (aprox 4°C) for at least one hour. After cooling, cold compress on the affected area for 1-2 minutes.



### Room Temperature :

#### **Recommended Indications for cold use :**

Optiben Dual Use Wipes can also use in room temperature daily for the hygiene of the eyelids and eyebrows.

Optiben Dual Use wipes presents a great physical and chemical compatibility with the lacrimal film and with balanced pH.

