

How to use POSIFORLID® Eyelid Cleansing Wipes?

1. If you have eye make-up, please remove your makeup before using POSIFORLID® Eyelid Cleansing Wipes.
2. Wash your hands before using POSIFORLID® Eyelid Cleansing Wipes.
3. Remove POSIFORLID® Eyelid Cleansing Wipes from the individual packaging.
4. Gently clean the eyelid with the closed eye by stroking from the outer to the inner side using POSIFORLID® Eyelid Cleansing Wipes.



5. Dispose of POSIFORLID® Eyelid Cleansing Wipes after use (do not flush it into the toilet).
6. Repeat the process on the other eye with a new POSIFORLID® Eyelid Cleansing Wipes.
7. Suitable for adults and children aged 3 or above.