How is POSIFORLID® Eyelid Hygiene Spray to be applied?

Before the first application, approximately 3-4 pumps are required so that it is absorbed. The Spray comes out atomized. At this point, the bottle is ready for all subsequent uses. The spray application options vary based on its use for the care of the eyelids or for the removal of any crusts. If you use eye cosmetics, it is recommended to apply the spray before applying make-up and/or after removing make-up from the eyes.

If you want to treat red and irritated eyelids, use POSIFORLID® Eyelid Hygiene Spray as follows:

- Hold the bottle at a distance of about 10cm in front of the eye.
- Close your eyes.
- Apply POSIFORLID® Eyelid Hygiene Spray with 1-2 sprays on the closed eyelids.
- Important: Please do not spray on the open eye. And do not touch the eye surface and eyeballs with the spray nozzle.

If you want to remove deposits on the eyelids or between the eyelashes and/or for the hygiene of the eyelid margins, there are two application methods:

1. Application with cotton swabs and spraying onto the eye.



- Moisten a cotton swab with POSIFORLID® Eyelid Hygiene Spray.
- Spray POSIFORLID® Eyelid Hygiene Spray onto the closed eye as described above.
- Massage the margin of the eyelid with the moistened cotton swab, first on the upper lid from top to bottom and then on the lower lid from bottom to top; each over the entire width of the eyelid.
- Then rub carefully from the outer to the inner evelid angle. The cotton swabs should only be used once.
- Finally, clean the margins of the eyelid with a moist cotton pad or POSIFORLID® Eyelid Cleansing Wipes.
- To clean the margin of the eyelid, always wipe the cotton pad on the eyelid from the outside to the inside.
- Suitable for adults and children aged 3 or above.

2. Application with a cotton pad.



- Operate the pump a few times to spray POSIFORLID® Eyelid Hygiene Spray onto a cotton pad until it is sufficiently moistened.
- To clean the margin of the eyelid, always wipe the cotton pad on the eyelid from the outside to the inside,
- Suitable for adults and children aged 3 or above.